

A. Procedure

ALS

1. Prepare equipment:
 - a) Nebulizer kit
 - b) Reservoir tube
 - c) Oxygen supply
 - d) Proper medication and dosage
 - e) Oxygen supply tubing
2. Place the correct amount of medication in the bottom half of nebulizer barrel (The bottom of the nebulizer can be identified by the oxygen connection port).
3. Connect the upper half of the nebulizer to the barrel.
4. Connect the "T" shaped directional tubing on top of the nebulizer barrel or connect barrel to a face mask.
 - a) Only use the aerosol face mask designed for the nebulizer.
DO NOT use ordinary oxygen masks.
5. Connect a mouthpiece to the "T" tubing.
6. Connect the reservoir tubing to the other end of the "T" tubing.
7. Connect oxygen supply tubing to the bottom of the nebulizer and oxygen supply.
8. Adjust the oxygen supply to 5 - 6 L/min (depending on the flowmeter and nebulizer specifications) to obtain a mist of medication. Only use non-humidified oxygen.
9. Nebulizer must be held upright for medication to be nebulized.